

Caution


NOTE: Always wear appropriate personal protective equipment (PPE) and follow the established professional protocols for pre-cleaning and disinfection. This quick start guide is only a summary of the steps necessary to operate the Knight FlushTech GI™. Be sure to follow the detailed steps outlined in the FlushTech Instruction Manual.

The FlushTech system requires daily decontamination of the internal pump and flush tubing. Consult your chemical supplier for appropriate disinfectant and requirements for proper high-level disinfection (HLD).


1 Decontamination Priming & Soak



A. Press and hold the Enter key until the FlushTech screen changes to the purple Decontamination screen.

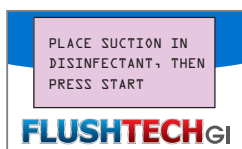

B. Insert the flush suction tube into the container or sink with the HLD solution.

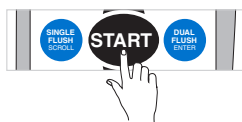

C. Press Start to prime the disinfectant solution through the pump and flush tubes. Allow the timer to count down.




Soak time will automatically start count down. An audible alert will beep when soak time is done.

2 Rinse




D. Insert the flush suction tube into the container or sink with fresh water and press Start to rinse the internal components with HLD solution. Allow the timer to count down.

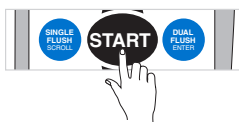

3 Rinse Purge



E. Remove the flush suction tube from the water and place it on the counter and press Start. The system will pull in air to purge the water from the internal components.

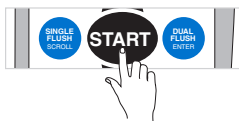

FOR MORE INFORMATION

(800) 999-2820
sales@cfstech.com

knightequip.com
cfstech.com